

Connecting Starkey, Signia or ReSound “Made for iPhone” hearing aids to Bluetooth (phone and media)

To pair your hearing device, follow these steps:

1. Make sure that Bluetooth is on. Go to Settings, then select Bluetooth.
2. On your Apple device, go to Settings > Accessibility > Hearing, and select Hearing Devices.
3. Open and Close the battery doors on your hearing device if you use disposable batteries, or place them in the charger and pull them back out if you use rechargeable aids. Your Apple device will search for your hearing aids.
4. Under MFi Hearing Devices, tap the name of your hearing device.
5. Tap Pair when you see the pairing request on the screen. If you have two hearing devices, you'll get two requests. Pairing could take up to a minute.